



# mi-eye 2<sup>™</sup>

The future of  
visualization has arrived





## See the answer like never before

Joint injuries can take weeks to visualize and diagnose with an MRI. Now, you can provide realtime analysis, effect faster treatment, and schedule patients for surgery immediately – in the comfort of your office. That is the power of mi-eye 2.

# mi-eye 2™



## Arthroscopy in the palm of your hand

mi-eye 2 is a handheld arthroscope, consisting of a needle with an integrated camera and a light source designed for in-office use.

- **120° field of view**
- **5—35mm depth of field**
- **Retractable 2.26mm needle**
- **Live image and video capture**
- **Powerful integrated optic and light source**
- **Ergonomic handle for one-handed/ambidextrous use**

**+** Covered by most major insurance carriers, including Medicare

#### FDA INDICATION FOR USE

mi-eye is indicated for use in diagnostic and operative arthroscopic and endoscopic procedures to provide illumination and visualization of an interior cavity of the body through either a natural or surgical opening

# The ultimate tool for surgeons

The Trice Tablet displays direct visualization from mi-eye 2 with striking precision. The performance of the Trice Tablet is unmatched by a traditional MRI's indirect imaging.\*



**+** See and diagnose joint injuries with incredible detail



HD Screen



10.8-inch display



Live video with playback



Save and share images and video



1.6GHz Intel Atom processor




Go-anywhere portability

\*Claim based on the 120° field of view and 35mm depth of field provided by mi-eye 2.

# The real value of a streamlined procedure

Before mi-eye 2, the diagnostic process took up to three visits before surgery or PT. Today, physicians can provide realtime analysis, and schedule patients for treatment after just one visit; getting them on the road to recovery faster.

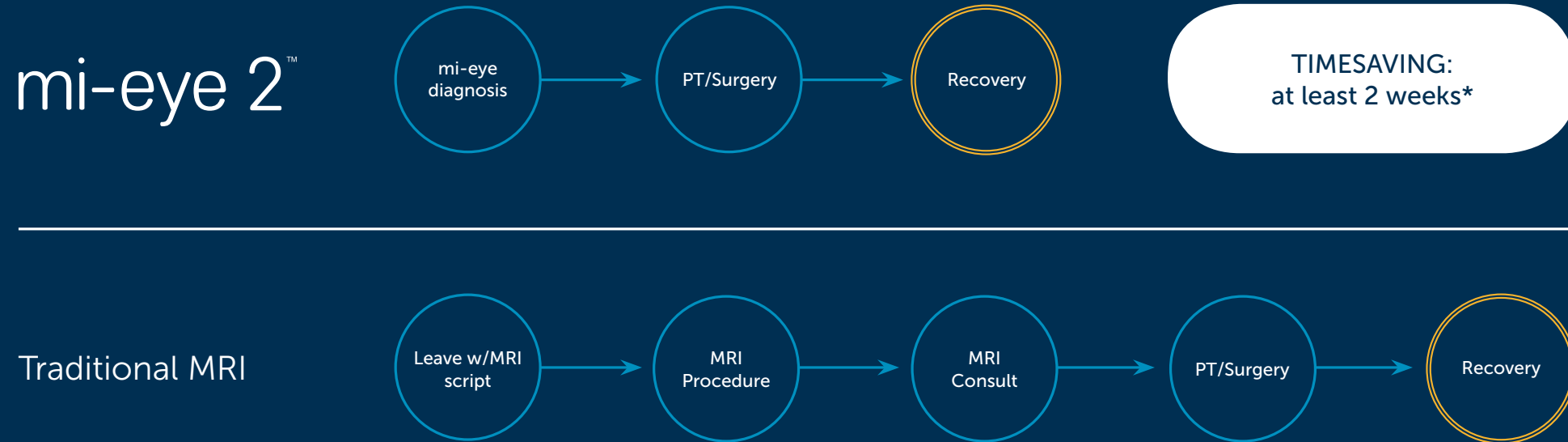


\*Based on estimated time-frame from radiology testing and results, including follow-up consultation.


# Patients are looking for immediate answers




86% value a diagnosis within the first office visit

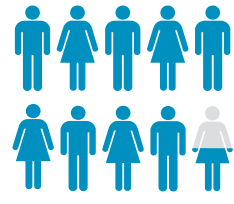

95% value real-time diagnosis over cost

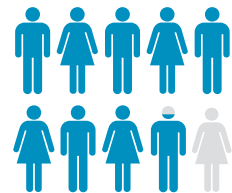

89% prefer mi-eye over MRI, given the benefits of both

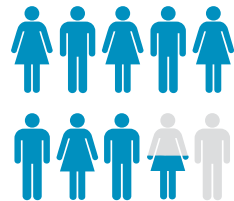

86% defer their treatment decision to their physician's recommendation

Based on 300 survey respondents

## Start the conversation with mi-eye 2

The revolution in orthopedic diagnostics begins in your office. mi-eye 2 puts your patients and your practice first. Learn more and discover technique guides, videos, and benefits at [tricemedical.com](http://tricemedical.com)



# Accessories & Supplies

## SMARTPHONE ADAPTER

Transfer your patient's mi-eye images and videos instantly into their hands with the Smartphone Adapter. Patients can leave your office and show their family and friends the mi-eye findings right on their Android or iOS-based smartphone.



## MI-PROBE

mi-probe allows physicians to percutaneously probe and palpate the anatomy to complete a full diagnostic arthroscopy of a patient's joint.

- **Sterile, disposable instrument**
- **Integrated, retractable trocar**
- **12mm blunted tip with six, evenly spaced markings**



## PROCEDURE KIT

Built to offer physicians the convenience of having the supplies needed for the performance of a mi-eye procedure, the mi-eye procedure kit includes items for:

- **Sterile field creation for supplies**
- **Portal preparation**
- **Sterile saline**
- **Post-procedure cleanup**
- **Portal coverage**



Local anesthetic and sterile gloves not included

# FAQs

## PROCEDURE

### **How will the integration of mi-eye™ affect my practice-flow?**

Trice Medical has effective best practices that can result in no interruption to your normal practice flow. Additionally, our Clinical Specialists will work with your practice to ensure the smoothest integration of mi-eye technology.

### **What additional supplies are needed to perform a mi-eye procedure?**

In general, the supplies needed are local anesthetic, gloves, aseptic preparation agent, syringes (with sterile saline), gauze and bandages. The mi-eye procedure kit is available for purchase and includes all items (excluding local anesthetic and gloves) in a box of 5 kits.

### **Who are the ideal mi-eye patients?**

mi-eye is ideal for patients that want an immediate answer to what is causing their pain. In addition, the mi-eye is ideal for patients that are not candidates for an MRI such as those with claustrophobia, pacemakers, metal implants or anxiety. mi-eye should not be used on a patient with an active infection.

### **Do patients have any issues tolerating a 2.2mm-gauge needle?**

Patients have tolerated the procedure very well, with most reporting little to no pain. It is important to note that every patient is different, and pain may be variable. We recommend an injection of local anesthetic prior to the mi-eye procedure to help eliminate any unnecessary pain or discomfort.

## REIMBURSEMENT

### **Will there be a reduction in reimbursement if I take a patient to surgery after mi-eye?**

If pathology that requires surgical intervention is discovered using mi-eye, your billing office can add the -58 modifier to the diagnostic code. This modifier will allow for reimbursement of all activities that occur during the 90-Day Global Period.

### **Do I need to get a pre-authorization before performing a mi-eye procedure?**

Prior-authorization is not a guarantee of payment. However, validated mi-eye procedural history has indicated that prior authorization is patient and plan dependent.

### **More Information**

The following CPT codes are used for mi-eye billing:

29805 - Shoulder Arthroscopy Dx  
29830 - Elbow Arthroscopy Dx  
29840 - Wrist Arthroscopy Dx  
29860 - Hip Arthroscopy Dx  
29870 - Knee Arthroscopy Dx

For more questions related to reimbursement please contact us at: 844.643.9300



<b>Part</b>	<b>Catalog Number</b>
95mm mi-eye	T200-095
160mm mi-eye	T200-160
Trice Tablet	T200-001
Smartphone Adapter	3-10-0137
mi-probe, 5-pack	T900-015
mi-eye Procedure Kit	T900-005
<i>Always refer to the products Direction for Use and User Manual for product information and approved use.</i>	